

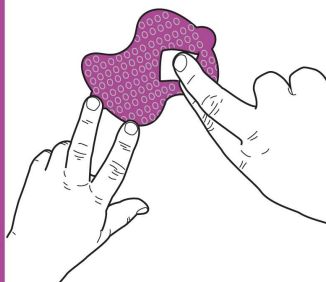
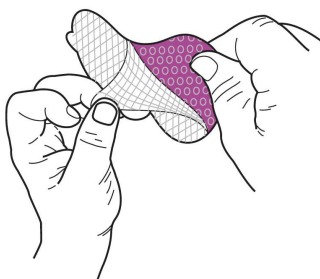


**1** Peel off used hydrogel from light patch. Remove the entire sheet and discard.

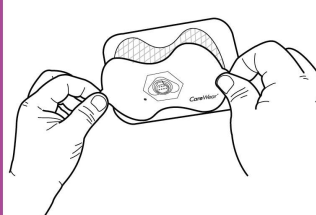


**2** Clean the light surface using a paper towel and water.

**3** Hold the hydrogel to the clear liner at the edge before peeling off the **blue** liner.

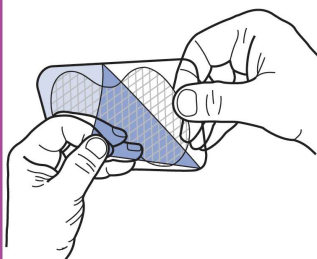

**4** Place the clear liner on a hard surface with the hydrogel facing up. Apply patch to hydrogel making sure alignment is correct.

**5** Apply pressure evenly over the patch surface. Press firmly around the entire patch edge.

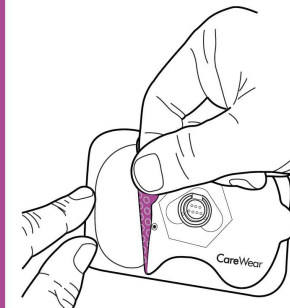

**6** Remove the light patch from the clear liner and place patch on clean, dry skin.