**HEINE** QUALITY  
MADE IN GERMANY

**HEINE UniSpec® Disposable Sigmoidoscope/ Proctoscope/Anoscope**

**CE**

HEINE Optotechnik GmbH & Co. KG  
Kientalstr. 7 · 82211 Herrsching · Germany  
Tel. +49 (0) 81 52 / 38 - 0  
Fax +49 (0) 81 52 / 38 - 202  
E-Mail: info@heine.com · www.heine.com

med 13496      2015-07-20

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##### DEUTSCH

## Instructions for use

	Please read and follow these instructions for use carefully and keep them for future reference.
--	---

#### Safety information

**Range of applications:** The HEINE UniSpec® disposable sigmoidoscopes, proctoscopes and anoscopes should be used once only. The should be fitted to the instrument head for UniSpec tubes with fibre optics illumination and used only for examination and treatment of the anal passage.

**Safety in use:** Use only light sources for the instrument head which have been approved for use in a medical environment. This tubes should only be inserted when an obturator is fitted and when attached to the instrument head.

#### 1. Preparing the Instrument

This is very simple, if the following order is observed:

Loosen the viewing screw (2) and open the window (1), which should be removed by extracting it upwards out of the instrument head.

Attach the instrument to the handle of the light source.

Insert the extension-piece (4) as far as possible into the instrument head and secure this by twisting to the right.

If required connect insufflation bulb to the window (5) and attach window to head.

Insert the obturator.

Apply a suitable lubricant to the tip of the obturator and the tube.

#### 2. Practical hints for use during the examination

The plane glass window (1) can be opened or closed at any time. The instrument can be made airtight by gently tightening the screw (2). If an insufflation bulb without a valve is used, the insufflated air can be regulated by gently loosening or tightening the screw.

If magnification of the object is required, insert the loupe into one of the two holes provided (3). The loupes are marked P for procto- and anoscope and R for recto- or sigmoidoscope. Both swivel to allow the window to be opened or closed. The telescope is inserted into both of the location holes (3) and clamped by means of a clamping screw. During instrumentation, the telescope can be folded downwards. (The telescope cannot be used with anoscopes).

	<b>Use of the Rubber Insufflation Bulb:</b> Always use the HEINE Hygiene Filter (8) when connecting the rubber insufflation (9) bulb to the HEINE Proctoscopes/Sigmoidoscopes (6). These filters are intended for single use only and prevent a contamination of the rubber insufflation bulb. For this, connect the transparent hose (7) to the insufflation port on the HEINE Proctoscopes/Sigmoidoscopes. Control the connection between the transparent hose and the HEINE Hygiene Filter. If it isn't securely attached, fix it more tightly. Then connect the Hygiene Filter to the rubber insufflation bulb. <b>Caution!</b> Infection! The use of the instrument without the HEINE Hygiene Filter may lead to contamination of the rubber insufflation bulb and thereby to an infection of the following patients. Therefore always use the filter. If you have forgotten to use the filter, replace the rubber insufflation bulb.
	<b>Risk of allergic shock:</b> The rubber double bellows contain natural rubber latex. Do not use it in patients with a latex allergy.

**Caution!** Infection! The filter is a single use product. If it is reused, the risk of infection increases. The filter is not suitable for being cleaned, disinfected and sterilized.

#### 3. Hygienic Reprocessing

- Instrument head**  
**Attention:** The polished end-surfaces of the optical fibers in the instrument head must be handled carefully to avoid damage to them which might lead to a reduction in light intensity.
  - 1.1 Cleaning**  
We recommend washing with a soft brush and soapy or plain water. If machine washing or chemical cleaners are used, the manufacturer's instructions regarding duration and concentration must be followed.  
**No ultrasonic cleaning.**  
**After cleaning, rinse thoroughly with demineralized water and dry at 65 °C max.**
  - 1.2 Disinfection**  
**We recommend soaking in disinfectant or thermochemical disinfection at up to 65 °C in a washer sterilizer.** Adhere strictly to manufacturer's instructions regarding concentration and duration.  
After disinfection, rinse with sterile water and wipe dry with a sterile cloth.
  - 1.3 Sterilization**  
Autoclaving can be used occasionally, but repeated autoclaving will reduce light transmission and life expectancy of the optical fibers.  
**Flash autoclaving and hot-air sterilization are not recommended.**
- Disposable tubes**  
The tubes are delivered clean and sealed in a transparent plastic sleeve. These tubes are for single use only. An attempt to clean them could result in damage to the tube or obturator and injury to the patient.
- Accessories**  
**Viewing window and loupes** can be sterilised by any normal method. **Rubber insufflation bulb** can be treated by wipe cleaning and wipe disinfection.

: Single use only

#### General Conditions of Warranty

Instead of the statutory warranty time period of 2 years, HEINE will grant a guarantee of 5 years from the date of the consignment of the goods ex works, concerning its equipment (excluding disposables, e.g. bulbs, single-use articles, and rechargeable batteries). Date of consignment means that HEINE hands the goods over to the transport carrier, freight forwarder or any other person designated by the Customer for the transport of the goods without loading the collecting vehicle. The guarantee covers irreproachable workmanship, on condition of the proper use of the equipment and the observation of the operating instructions. During the warranty- and guarantee time period, errors and deficiencies arising on the equipment will be rectified free of charge, in so far as such are evidenced by defective materials, processing and/or constructional errors. Should buyer complain of a material deficiency during the warranty time period, then the onus of proof is always to be on the orderer, that the product was defective already upon receipt of the goods. The statutory warranty and the guarantee do not apply to loss or damage caused by wear and tear, negligent use, the non-employment of original HEINE components and/or spares (in particular bulbs, as these have been especially developed for HEINE instruments in accordance with the following criteria: colour temperature, useful service life, safety, optical quality and performance). The statutory warranty and the guarantee do not apply to interventions by persons not authorised by HEINE or when the operating instructions are not observed by the customer. Any modification of a HEINE product with parts or additional parts which do not conform to the original HEINE specification will invalidate the warranty for the correct function of the product and further invalidate any warranty claims which result from such a change or modification. Further claims, in particular claims for replacement of loss or damage, which are experienced otherwise than directly on the HEINE product itself, are hereby excluded.

<hr/>	<hr/>
Bought from	
<hr/>	<hr/>
date	
<hr/>	<hr/>

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